




# PRODUCT CATALOG

# CPAP SOAP BUBBLE PADS AND SOAP PACKETS

SKU: SOAP90, SOAP180, SOAP365

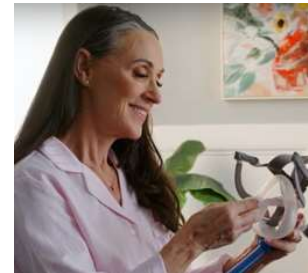

CPAP Soap provides convenient, consistent, and time-saving care for CPAP equipment, ensuring peace of mind and proactive maintenance for a more effective and hygienic therapy experience. It adheres to CPAP manufacturer guidelines and FDA-approved processes for recommended daily and weekly cleaning.

- HSA and FSA approved
- Alcohol Free
- SLS Free
- Non-allergenic
- Made in the USA



SCAN FOR VIDEOS

## HOW IT WORKS

Use CPAP Soap Bubble Pads for DAILY cleaning of CPAP accessories. After wiping down product, discard the pad and let the item(s) air dry. Mix one 5ml CPAP Soap Packet with a gallon of water for weekly washing of CPAP equipment. Rinse equipment and let air dry. Omit using Bubble Pads on Soap Packet cleansing days.

## CPAP SOAP PACKAGE OPTIONS

SOAP90: 90 DAY SUPPLY

SOAP180: 180 DAY SUPPLY

SOAP365: 365 DAY SUPPLY



## WHAT MAKES CPAP SOAP DIFFERENT

1

*CPAP Soap* is designed to be compatible with the materials used in CPAP equipment and crafted to effectively remove residues without causing damage. The components of CPAP equipment, such as masks, hoses, and reservoirs, are often made from delicate materials like silicone and plastics.

2

*CPAP Soap Bubble Pads* are recommended for daily use to remove physical particles from the face mask, while *CPAP Soap* packets should be used weekly to wash the mask, hose, and water chamber. *CPAP Soap* is SLS-free, alcohol-free, fragrance-free, and non-allergenic. *CPAP Soap* adheres to FDA-approved processes and allows patients to follow manufacturer guidelines to ensure the safest and most effective formula for washing CPAP equipment.

3

*CPAP Soap* does not contain SLS (Sodium Lauryl Sulfate). SLS is a foaming agent found in many soaps and detergents that can irritate the skin by stripping away the upper dermis layer. This can result in symptoms like redness, dryness, itchiness, and peeling. Some mild detergents can contain SLS, fragrances, and other chemicals that can irritate patients with increased sensitivities.

4

*CPAP Soap* does contain SLES, or Sodium Laureth Sulfate, which is a milder derivative of SLS that doesn't remove skin layers or provoke irritation. It's often used as a gentler alternative to SLS in various products such as soaps, detergents, cleaning agents, and makeup removers.

5

*CPAP Soap* is fragrance free. Fragrances are frequent culprits in causing contact allergies, leading to issues like dermatitis, dryness, and sensitivity.

6

*CPAP Soap Bubble Pads* are alcohol free. Although alcohol wipes are popular, they can potentially be harmful to CPAP equipment due to their potential to degrade certain materials. Additionally, alcohol may not effectively remove all types of residue or buildup from CPAP equipment, especially oils and greases from the skin. This residue can accumulate over time and contribute to bacterial growth and/or unpleasant odors.