

ResMed

AirFit™ F30i


Full face mask

Actual Size: 9 3/4" (Width) x 4" (Height)



Ruler is for scaling purposes.

How to use the fitting template:

1. Please ensure the template is printed to scale
2. Cut out along dotted line
3. Refer to diagram on template to determine appropriate cushion size
4. Refer to diagram on right to determine appropriate frame size

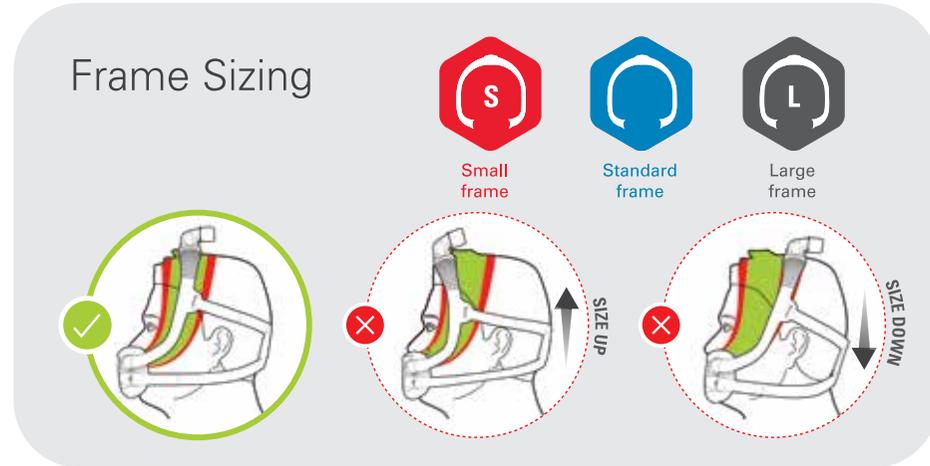

Cushion Sizing

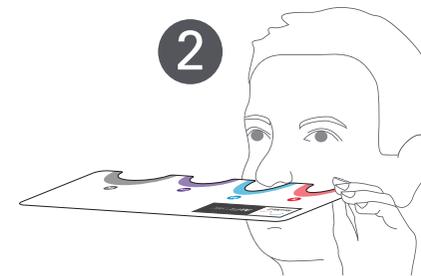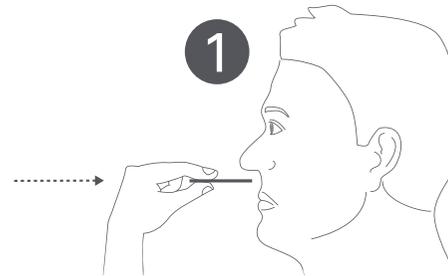


S

M

SW

W



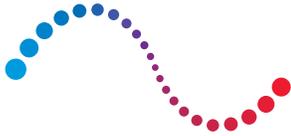

ResMed

AirFit™ F30i

Fitting guide for AirFit F30i



- With one hand holding the bottom of the cushion and the other holding the conduit frame, position the mask in front of the face.



- Place the cushion under the nose and against the face. With the ResMed logo and headgear facing up, pull the headgear over the head.



- Bring the lower headgear straps under the ears and attach the magnetic clips to the frame connectors.



- Undo the fastening tabs on the upper headgear straps and pull evenly.



- Repeat with the lower headgear straps.



- Squeeze the side buttons on the elbow and detach from the frame. Connect the air tubing from the device to the elbow, then reattach it to the frame.



For people with long hair, we recommend feeding the hair through the back of the headgear, as pictured. If tied in a ponytail, it should sit high enough to fit through the back of the headgear.

Refer to the user guide for further instructions and complete labeling information including indications, contraindications, warnings and precautions.

* Calculations based on CPAP users in the study who had a mask preference. CPAP users rated objective criteria for AirFit F30i and DreamWear full face on a Likert Scale of 0-10. Statistically significant differences ($p < 0.05$) in median scores were found between the two masks. Easy to use is defined as: easy to fit, adjust and remove.

1 ResMed guided external clinical study of 48 evaluable current full face CPAP mask users from Oct. 24 to Nov. 20, 2018 in Sydney, AU and San Diego, CA, who trialed AirFit F30i and Phillips™ Respiroics DreamWear full face mask at home.

Distributed by ResMed Corp, 9001 Spectrum Center Boulevard, San Diego, CA 92123 USA +1 858 836 5000 or 1 800 424 0737 (toll free). See ResMed.com for other ResMed locations worldwide. AirFit is a trademark and/or registered trademark of the ResMed family of companies. DreamWear and Phillips are trademarks of Koninklijke Philips N.V, registered in the US and other countries. Specifications may change without notice. For patent and other intellectual property information, see ResMed.com/ip. © 2019 ResMed. 10111403/2 2019-11